

LinAqua forte, solution for nasal cavity irrigation. Hypertonic concentration, for children and adults

Description and composition

LinAqua forte is odorless and colorless solution for nasal cavity irrigation.

Natural sea water contains sodium chloride (NaCl) of 21 g/l (hypertonic concentration).

Contains all active substances and trace elements of sea water: K, Mg, Na, Ca, Cl, Se, I, Zn, Cu, Fe, etc. Does not contain preservatives.

Therapeutic indications

Otorhinolaryngology. The solution is intended for everyday nasal hygiene and as a part of complex treatment of ENT diseases for children from 2 years and adults.

Properties

Solution provides moisturizing effect and relieves irritation. Thoroughly rinses all parts of the nasal cavity and nasopharynx, flushing out bacteria, viruses, allergens, crusts, mucus, dust particles.

Maintains the normal physiological state of the nasal mucosa, boosts local immunity.

Reduces swelling of the mucous membrane of the nasal cavity and nasopharynx, helps to restore nasal breathing.

Helps to increase the therapeutic efficacy of medications applied in the nasal cavity and reduces the duration of respiratory diseases.

Reduces the risk of spreading the infection to the paranasal sinuses and ear cavity.

Accelerates the processes of regeneration of the mucous membrane of the nasal cavity and nasopharynx, reduces the risk of complications after surgery in nasal cavity and paranasal sinuses.

Indications for use

For children from 2 years and adults:

- daily use as a part of complex treatment of acute and chronic inflammatory diseases of the nasal cavity: acute and chronic rhinitis; acute and chronic sinusitis; acute and chronic adenoiditis; allergic and atrophic rhinitis;
- to prepare the nasal cavity for nasal medications use;
- to wash out cavity before surgery

Contraindications for use

- individual intolerance to the components of the solution;
 - complete obstruction of the nasal passages;
 - acute otitis and exacerbation of chronic otitis.
- If in doubt consult your doctor or ENT specialist.

Precautions for use

Inspect the packaging and product before use. Do not use the product if the packaging is damaged.

Avoid contact with eyes.

Possible side effects

No side effects have been reported.

Interactions with other medicinal products and medical devices

Can be combined with all nasal products and physiotherapeutic procedures.

Recommended dosage

As a part of complex treatment and prophylaxis: flush out nasal cavity 4 times a day.

The duration and frequency of use of the product is not limited.

Recommendations for use

- Tilt your head to one side
- Insert the nozzle into one nostril
- Press the nozzle for several seconds, then blow your nose
- If necessary, repeat the procedure
- Repeat in the other nostril



Clean the spray tip and put on a protective cap after use.

Special precautions

If you are pregnant or breastfeeding consult your healthcare specialist before use.

It does not interfere with the ability to drive or use machinery.

Requirements for use

The product is for individual use only. Using at home does not require special training and special skills.

Contents of the pack

Aerosol bottle with a jet spray (volume of 50 ml, 100 ml, 125 ml, 150 ml or 200 ml) and with a nozzle for nasal cavity.

Shelf life

3 years. Do not use after expiration date.



Manufacturer / For noncompliance please contact:

Grotex LLC,
Russian Federation, 195279,
Saint Petersburg,
Industrialniy av. 71, bld. 2, lit. A
Tel.: + 7 812 385 47 87
Fax: +7 812 385 47 88
www.solopharm.com
www.linaqua.ru



Storage conditions

Store at temperature from + 10 °C to + 30 °C, keep out of reach of children. Keep away from heat. Keep out of direct sunlight and source of open flame.

